

3! | POB-MA

FLEX | HYDROPHOBIC PRELOADED

3! | FLEX HB

oerli
ophthalmedic®


www.oerli-ophthalmedic.ch



EYE SURGERY ESSENTIALS.



EASY, SAFE AND
PREDICTABLE
INJECTION SYSTEM

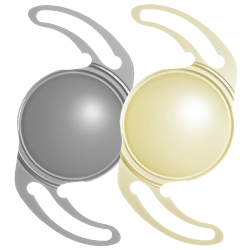

1st STEP Fill visco (HPMC) through the aperture until the optic of the lens is covered

2nd STEP Keep the injector at this state for a minimum of 3 minutes

3 mins.

3rd STEP Rotate the transparent ring until you hear a click to close the cartridge

4th STEP Pull out the red stopper

5th STEP Remove the ring-adaptor from the injector

INJECTION Push the plunger continuously and do not pause until the optic exits the cartridge tip

For detailed information please consult Instructions for Use (IFU) of the product.

Lens types	Single-piece Aspheric Hydrophobic Acrylic IOL
Material	Hydrophobic Acrylic with UV Absorber + optional blue light filter
PCO protection	360° Special Square Edge
A-Constant**	118.9 (SRK/T)

Diameters	6.0 mm (optic) / 13.0 mm (overall)
Haptic angulation	0° – posterior vaulting fenestrated C-loop
Powers available***	-10.0D → +9.0D · (increment: 1.0 D) +10.0D → +30.0D · (increment: 0.5 D) +31.0D → +35.0D · (increment: 1.0 D)

* patent protection

** it is recommended that surgeons personalize the constants they use

*** other powers upon request